

LINQ II™ LNQ22

Insertable Cardiac Monitor

Insertion Procedure Quick Reference Card




© 2018 Medtronic
M974774A001 A
2018-12-12



Medtronic



Medtronic

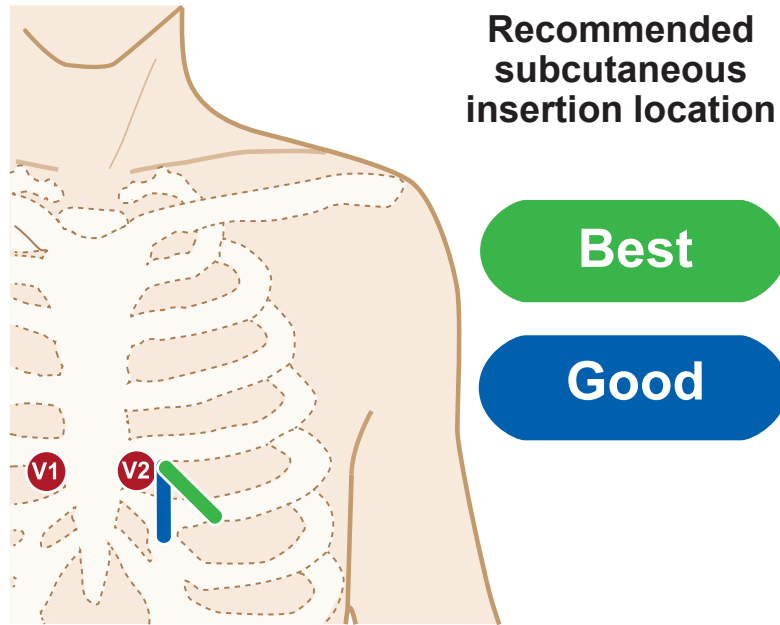


Recommended subcutaneous insertion location


Best

Good

Note: For optional insertion locations and the detailed insertion procedure, refer to the LINQ II Clinician Manual.

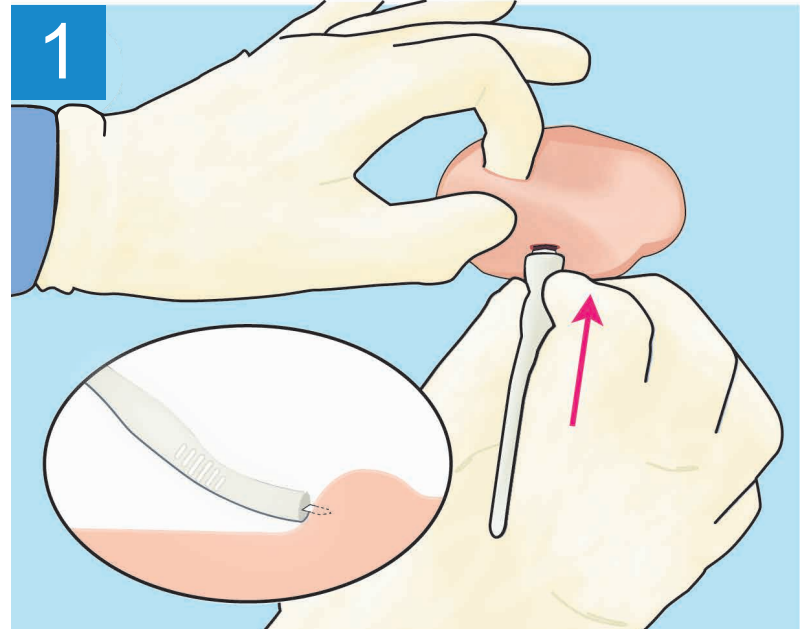


Pinch the skin and make an incision at an angle of 90 degrees with the supplied incision tool.

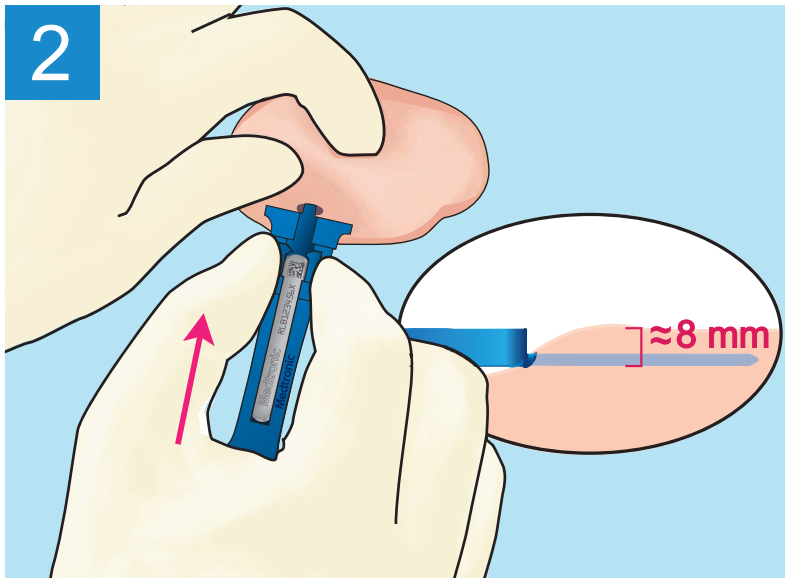


Pinch the skin and insert the supplied insertion tool completely to create a tunnel approximately 8 mm deep under the skin.



Rotate the insertion tool 180 degrees once to open the incision for inserting the cardiac monitor.

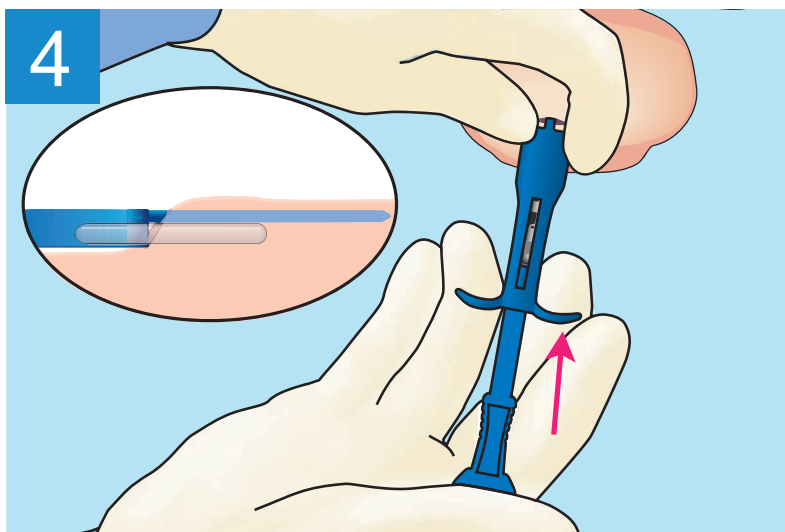

Insert the supplied plunger to push the preloaded monitor completely into the pocket. When the plunger is completely pushed into the insertion tool, the monitor is fully inserted.

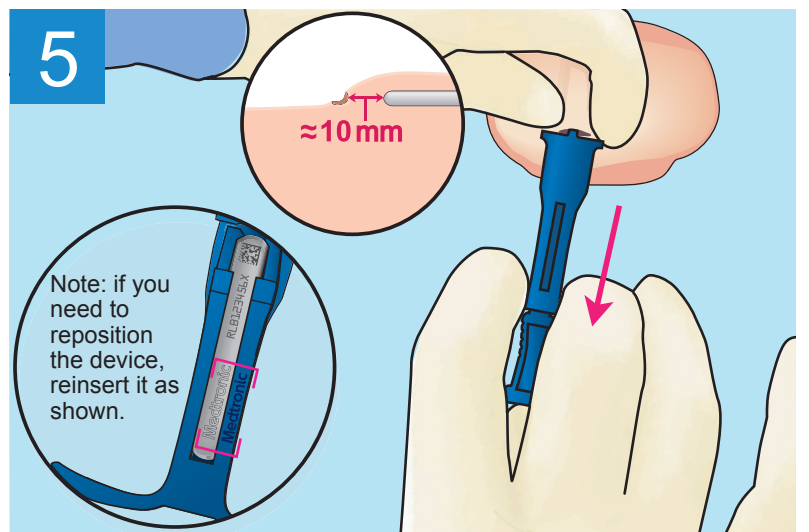

Pinch at the insertion site to hold the monitor in place (approximately 10 mm past the incision). While holding the monitor in place, remove the plunger and the insertion tool. Close the incision site using your preferred method.

